

INSTRUCTIONS

Setting Up Your Safe Place Bed



Safe Place
Bedding LLC



Youtube video:
Basic setup of the Safe Place Bed

1. Each Safe Place Bed model 100 consists of a canvas cover with four inflatables inside, four straps, and instructions.

2. Place the Safe Place Bed on a stationary bed, which is at least 135cm wide, and unfold it. Ensure the netting on the door closest to you is fully visible.




3. Pull a zipper tab out of the canvas pocket and open zip until it is 2-3cm above the valve on the inflatable (do not completely open the zip or it will be difficult to close after inflation) Ensure the black valve is fully inserted into the clear opening with no gap.



4. Repeat for all four sides of the Safe Place Bed

5. Inflate all four sides of the Safe Place Bed. You may need to hold the air pump in black valve. The Safe Place Bed should be erect.

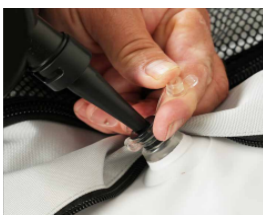

6. To seal the inflatable, insert the clear plug into the black one-way valve, then press the valve into the inflatable chamber. Repeat for all four valves. Zip all four chambers closed push zip tab back into the canvas pocket.




7. Put the Safe Place Bed in the center of the stationary bed. There should be at least 10cm of space between the long edges of the Safe Place Bed and edge of the mattress. Refer to the picture for guidance. A full-length bed guard must be used if the stationary bed is less than 135cm wide.



8. Connect two straps to the long side of the Safe Place Bed. Pass the straps underneath the stationary bed and fasten on the opposite side of the Safe Place Bed. Tighten the straps – see step 9 for details.

Connect two straps to the short side of the Safe Place Bed. Pass the straps underneath the stationary bed and fasten on the opposite side of the Safe Place Bed. Tighten the straps. The straps should form a "hashtag #" underneath the stationary bed.

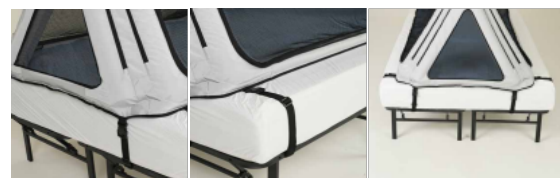

YouTube video: folding the bed

9. Remove slack from each of the four straps until the Safe Place Bed is firmly attached to the stationary bed. Pull strapping up in the middle of the buckle, taking slack away above the buckle. To lock strapping in place, pull the tail of the strapping



*Note that there are two buckles on each strap.

Not tightening straps can result in serious injury. During every use, please inspect strapping and buckles for damage.

Zip lock:

Connect the clasp to the zip tab and tighten the fabric strap until there is no gap between the zipper tab and the cloth. When inflating and deflating the bed, please ensure the doors are closed and locked.



Daily checks while using the Safe Place Bed

Before using the Safe Place Bed, check for imperfections in the mesh.

- Squeeze all four inflatables to ensure they are still inflated. If one chamber is deflated, remove clear plug and black one-way valve from clear plastic opening, clean, dry, replace and reinflate. Do not use bed for 24 hours to ensure inflatable will hold air. If it deflates again, contact customer service.
- Pull all four straps to ensure each strap is tight and check the Safe Place Bed is in center of the stationary bed.
- Check the zip works correctly as you close the door.
- Check zipper lock and tighten if necessary.
- If there are problems with any of the above, do not use the Safe Place Bed and contact customer service.

Cleaning:

- Clean bed with warm water. Do not use laundry detergent because it will remove the waterproof coating on the fabric.

Changing an inflatable chamber:




YouTube video:
Changing the chambers

- Open both zips and deflate inflatable.
- Pull the inflatable out of the canvas cover.
- Reverse process to replace inflatable chamber, making sure the valve point outwards.



To Change Long Chamber (rectangle):

- Open both zips fully and deflate inflatable.
- Push inflatable chamber along the top of the canvas cavity.
- Move to the opposite side of the Safe Place Bed and pull inflatable chamber through
- Reverse process to replace inflatable chamber, making sure the valve point outwards.

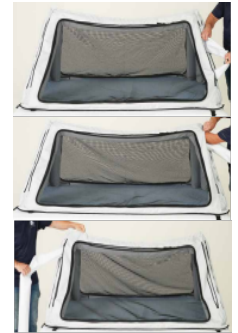

Warnings:

Not for children under 36 months due to small parts and risk of suffocation.

No smoking or heat sources around products.

Keep away from water, not a flotation device.

Not a toy.

If one side of bed deflates, stop using bed.

Keep sharp objects away from product, punctures are not covered by warranty and may cause injury to child.

- Firmly attach to stationary bed when in use, not tested independently.
- The Safe Place Bed is 105cm wide and 190cm long and must be used on a stationary bed which is at least 135cm wide and 190cm long.
- The zip lock must be tightened before use and then checked nightly to prevent entrapment.
- Monitor user closely when the user or others have access to the outside of the Safe Place Bed, risks may include but are not limited to strangulation from straps, choking on zip and entrapment.



To change end chamber (triangle)