




SAFE PLACE BED INSTRUCTIONS



BEFORE FIRST USE: USE A DAMP CLOTH TO GENTLY WIPE THE CLEAR, BLACK, AND WHITE PARTS OF EACH VALVE.

STEP 1

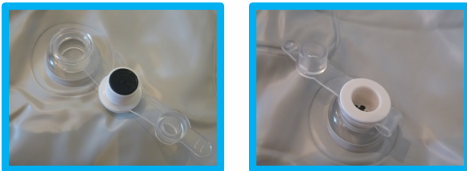

Inflate and deflate the Safe Place Bed with zippers closed and locked.
Place the Safe Place Bed on a stationary bed that is at least 190 cm long and 135 cm wide.
Insert the white valve into the clear plastic hole. Make sure it is fully inserted. Repeat for all four valves.

STEP 2



Inflate all four sections of the Safe Place Bed. Each inflatable part of the bed should be firm, but not hard.
Be careful not to inflate the bed too much, this will void the warranty!

STEP 3



Push the clear plug all the way into the white valve, then slide the entire valve into the Safe Place Bed.
Repeat for all four valves

STEP 4



Check that the Safe Place Bed is in the middle of the stationary bed. Connect a white strap to a white clip. Pass the strap under the stationary bed and connect the white clip on the other side of the Safe Place Bed.
Repeat at the opposite end of the Safe Place Bed. Tighten the white straps until the Safe Place Bed cannot move or tip over.

ZIPPER LOCK



Close the zippers at the end of the bed with the zipper clip. Attach the clip to both zippers and pull the strap tightly so the zippers cannot be moved.

DAILY INSPECTION OF THE SAFE PLACE BED BEFORE USE

1. Inspect the Safe Place Bed every day before placing the child in it:
2. Perform a visual inspection to check that the Safe Place Bed is in the middle of the stationary bed and that the mesh is intact.
3. Perform a manual inspection:
 - i. Squeeze each part of the bed to ensure it is still inflated. If a section has emptied of air, clean all three sections of the valve, and inflate again. Do not use the bed for 24 hours to ensure that the section holds air. If the section deflates again, contact safeplacebedseu@gmail.com
 - ii. Pull each white strap by hand to ensure it is tight and the Safe Place Bed remains in the middle of the stationary bed.
 - iii. Pull on the zipper when closing the Safe Place Bed.
4. if you see a problem, do not use the bed, and contact safeplacebedseu@gmail.com

STAY SAFE WHILE USING YOUR SAFE PLACE BED

1. Inspect the safe for damage every time you use it.
2. Talk to your doctor before using a Safe Place Bed if your child is under 36 months of age or has low muscle tone. Risk of suffocation.
3. Please monitor the child/user closely when accessing the outside of the Safe Place Bed. The risk may include, but is not limited to, suffocation from straps, suffocation on the zipper.
4. No smoking or heat sources near Safe Place Bed
5. Keep the Safe Place Bed away from water, it's not a flotation device.
6. Keep sharp objects away from the Safe Place Bed, punctures are not covered by the warranty and may harm the child.
7. This product is not a toy.
8. Always consult your doctor before using this product

Contact us:

www.safeplacebedseu.com

safeplacebedseu@gmail.com

+447782105085